HeraCeram Zirkonia

(DE)	Gebrauchsanweisung	3
(GB)	Instructions for use	6
(FR)	Mode d'emploi	9
(ES)	Instrucciones de uso	12
(IT)	Istruzioni per l'uso	15
(PT)	Instruções de uso	18
(NL)	Gebruiksaanwijzing	21
(SE)	Bruksanvisning	24
(DK)	Brugervejledning	27
(NO)	Bruksanvisning	30
(FI)	Käyttöohjeet	33
(GR)	Οδηγίες χρήσης	36
(CZ)	Návod k použití	39
(HU)	Használati utasítás	42
(LV)	Lietošanas instrukcija	45
(LT)	Naudojimo instrukcija	48
(PL)	Instrukcja obsługi	51
(HR)	Upute za uporabu	54
(RO)	Instrucţiuni de utilizare	57
(TR)	Kullanma talimatı	60
(AE)	تعليمات الاستخدام	63
(KR)	사용 설명서	66
(SK)	Návod na použitie	69
(BG)	Инструкции за употреба	72
(SI)	Navodila za uporabo	75
(RS)	Uputstva za upotrebu	78
(MK)	Упатство за употреба	81

MD

Medizinprodukt
Medical device
Dispositif médical
Producto sanitario
Dispositivo medico
Equipamento para saúde
Medisch hulpmiddel
Medicinteknisk produkt
Medicinsk udstyr
Medisinsk utstyr
Lääkinnällinen laite
Ιατροτεχνολογικό προϊόν
Zdravotnický prostředek
Orvostechikai eszköz
Medicīniska ierīce
Medicinos priemonė
Wyrób medyczny
Medicinski proizvod
Dispozitiv medical
Tibbi cihaz
جهاز طبي
의료기기
Zdravotnícka pomôcka
Медицинско изделие
Medicinski pripomoček
Medicinski proizvod
Медицински производ

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Stand: 2022-03

Medical Device – for use by dental health care professionals only.

Intended purpose

Dental ceramic for veneering, customisation and modification of prosthetic restorations. Classification according to EN ISO 6872: Type I; Class 1

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

Indications

Veneering ceramic for frameworks made of zirconium dioxide, CTE/ coefficient of thermal expansion (α25–500°C) 10.5 µm/m·K.

Contraindications

- Bruxism or other parafunctional habits.
- The use of this product is contraindicated in case of known or suspected allergies against components of this product.

Potential side effects

Hypersensitivities to the product or its components cannot be excluded in individual cases.

Composition

- **HeraCeram Zirkonia powder materials**
Glass ceramic, tin(IV) oxide, chromium-tin-pink-spinel, vanadium-zirconium-yellow-baddeleyite, chromium-tin-pink-sphene, diyttrium oxide silicate
- **HeraCeram Zirkonia liner (paste)**
Glass ceramic, 1,4-butanediol, zirconium praseodymosilicate, zinc-iron-chrome-brown-spinel, D-panthenol, chrome-tin pink sphene, cerium(IV) oxide, diyttrium oxid silicate
- **HeraCeram Zirkonia Zr Adhesive / Zr Adhesive Neutral (paste)**
Glass ceramic, 1,4-butanediol, D-panthenol

Application notes

- HeraCeram Zirkonia must not be mixed/combined with veneering ceramic components of ceramic lines manufactured by competitors.
- The use of modelling/staining liquids or thinner liquids of ceramic lines manufactured by competitors is not indicated.
- Powdery materials (e.g. dentine, incisal, etc.) must not be mixed with paste-like materials (e.g. adhesive, liner, stains, glaze).
- If the consistency of the HeraCeram Zirkonia Zr-Adhesive and liner paste becomes drier and thus firmer due to prolonged storage, the viscosity can be readjusted by carefully adding the HeraCeram Paste Opaque Liquid (POL).
- Frameworks for ceramic veneering should have the reduced anatomical shape. Sharp edges and corners on the frameworks should be avoided.

Product application

1. Framework preparation

After completion of framework processing, the items must be thoroughly cleaned of abrasive dust and abrasive residues. The zirconium oxide frameworks must be cleaned with the steam jet. It is not necessary to blast the veneer surface to increase the adhesive bond.

2. Adhesive

For an optimum bond between zirconium dioxide and HeraCeram Zirkonia, HeraCeram Zirkonia Zr-Adhesive paste is applied to the framework surface with a brush in an evenly thin layer (wash) and fired under vacuum (see firing chart).

3. Liner application

If necessary, a HeraCeram Zirkonia liner can also be applied in an evenly thin layer and fired to cover the framework and provide base colouration. This step can be repeated if the liner does not cover it completely. Alternatively, the HeraCeram Zirkonia increasers can also be used for this base layer. Firing procedure see firing table.

4. Layering

The ceramic veneer is designed in a two-layer technique (dentine incisal layering) or, depending on the desired aesthetic requirements, with additional individual materials², and is fired. The margin ceramics LM and correction material (COR) are used for corrections after completing the veneering. The margin ceramics HM are used to produce all-ceramic crown margins. To produce the ready-to-use state, the ceramic powders are mixed with the modelling liquids ML or MLS from Kulzer's HeraCeram range. HeraCeram Zirkonia margin ceramics HM and LM are mixed with HeraCeram margin ceramic liquid (SML). For the firing procedure, see the firing table.

Note: To prevent breathing in the ceramic dust when grinding the ceramic materials, wear a face mask and eye protection and work with a suction device.

5. Glaze firing

The veneer can be individually characterised with HeraCeram Glaze universal and HeraCeram Stains universal stains according to the Instructions for Use. The degree of glazing and the texture of the ceramic surface are defined during glaze firing by the temperature and the holding time until the final temperature. For this reason, the specifications for the glaze firing temperature and holding time are for orientation only and may need to be adjusted to achieve the required result. Firing procedure see firing table. It is also possible to mechanically polish HeraCeram Zirkonia.

Colour mapping table

	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4	BL1	BL2	BL3	BL4
Liner	L A1	L A2	L A3	L A3,5	L A4	L B1	L B2	L B3	L B4	L C1	L C2	L C3	L C4	L D2	L D3	L D4	L BL 1-2	L BL 1-2	L BL 3-4	L BL 3-4
Increaser	IN A1	IN A2	IN A3	IN A3,5	IN A4	IN B1	IN B2	IN B3	IN B4	IN C1	IN C2	IN C3	IN C4	IN D2	IN D3	IN D4				
Dentine	D A1	D A2	D A3	D A3,5	D A4	D B1	D B2	D B3	D B4	D C1	D C2	D C3	D C4	D D2	D D3	D D4	D BL1	D BL2	D BL3	D BL4
Incisal	S1	S1	S2	S2	S4	S1	S1	S2	S4	S1	S3	S3	S3	S1	S2	S2	S BL	S BL	S BL	S BL
Mamelon secondary dentine	MD 1	MD 1	SD 2	SD 3	SD 4	MD 2	MD 2	MD 3	MD 3	MD 2	SD 1	SD 2	SD 2	MD 1	MD 3	SD 1				
Value	VL 1	VL 2	VL 3	VL 4	VL 4	VL 1	VL 2	VL 3	VL 4	VL 1	VL 2	VL 3	VL 4	VL 2	VL 3	VL 4				
Opal incisal	OS1	OS1	OS2	OS2	OS4	OS1	OS1	OS2	OS4	OS1	OS3	OS3	OS3	OS1	OS2	OS2	OSBL	OSBL	OSBL	OSBL
Margin ceramics HM/LM	1	1	2	2	6	3	3	4	4	5	5	6	6	1	2	4	HM / LM BL1-2	HM / LM BL1-2	HM / LM BL3-4	HM / LM BL3-4

General firing table

IMPORTANT NOTICE: The firing temperatures given are intended as guidelines. Deviations are possible due to varying furnace power and may need to be adapted.

		Zr-Adhesive	Liner	1. Margin ceramics HM	2. Margin ceramics HM	1. Dentine ²	2. Dentine ²	Stains-/Glaze universal	Correction material COR	Margin ceramics LM
Start temperature	°C	600	600	600	600	600	600	600	600	600
Drying	min	6	6	4	3	5	5	5	4	4
Pre-heating	min	1	1	2	2	2	2	1	2	2
Temperature increase	°C/min	100	100	100	100	100	100	100	100	100
Final temperature	°C	1050	880	870	860	860	850	850	810	790
Holding time	min	10 ¹	1	1	1	1	1	0,5-1	1	1
Vacuum start	°C	600	600	600	600	600	600	-	600	600
Vacuum stop	°C	1050	880	870	860	860	850	-	810	790

Cooling rate: There are no defined cooling rates required for HeraCeram Zirkonia veneering ceramics.

¹ = under vacuum

² implies the following masses: Dentine / Incisal / Transpa / Mamelon secondary dentine / Value / Opal incisal / Opal transpa / Enhancer / Mask / Gingiva / Increaser

Technical data

Dentine / Incisal / Transpa / Mamelon secondary dentine / Value / Opal incisal / Opal transpa / Enhancer / Mask / Gingiva / Increaser:

CTE (25-400°C)	9,5 ± 0,2 µm/m·K
CTE (25-500°C)	9,7 ± 0,5 µm/m·K
TG	510 ± 20 °C

Correction material COR / Margin ceramics LM:

CTE (25-400°C)	9,4 ± 0,3 µm/m·K
CTE (25-500°C)	not available
TG	515 ± 20 °C

Margin ceramics HM:

CTE (25-400°C)	9,5 ± 0,2 µm/m·K
CTE (25-500°C)	9,6 ± 0,4 µm/m·K
TG	550 ± 20 °C

Zr-Adhesive:

CTE (25-400°C)	9,6 ± 0,4 µm/m·K
CTE (25-500°C)	9,6 ± 0,4 µm/m·K
TG	640 ± 20 °C

Liner:

CTE (25-400°C)	9,6 ± 0,2 µm/m·K
CTE (25-500°C)	9,4 ± 0,25 µm/m·K
TG	540 ± 20 °C

If the glass transition temperature is approximately 500 ± 30°C and there is a significant change in slope of the CTE curve before and after the glass transition temperature, then the value of 400°C is stated.

General information

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself. Visually inspect for damage before use. Damaged products must not be used.

Warnings and safety instructions

Avoid breathing dust/fume/gas/mist/vapours/spray. Ensure adequate ventilation and exhaust. Wear suitable face mask and eye protection. Inhalation can cause health problems. May cause drowsiness or dizziness. These dangers and safety instructions do not refer to the prosthetic restoration itself, but to the vapours and dusts released during the processing of the material.

Special advice

For SSCP please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact sscp-service@kulzer-dental.com
To find the SSCP in the EUDAMED use the UDI code attached at the packaging of the product.

Storage conditions

Carefully reseal the container after removing the product to avoid contamination, clumping and drying out. No special storage conditions required.

Disposal information

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system.

European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Complaint notice

Please quote batch number [LOT] and article number [REF] or UDI in all correspondence about the product.

According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Safety data sheets and more information are available at our website www.kulzer.com

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Kulzer GmbH